M•PRODUCTS

mobility solutions




VIRT 2

Quick Manual

Version 1.0 Controls

Instruction	Date:
Name:	
Seen	

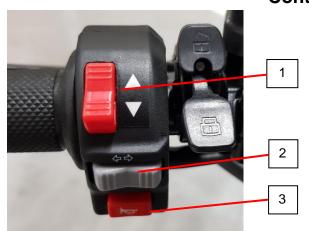



Controls - Left handlebar switches

- 1. Forward / reverse switch
- 2. Direction indicator left / right
- 3. Horn



Left handlebar

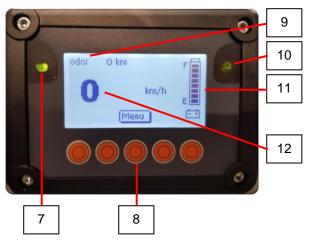

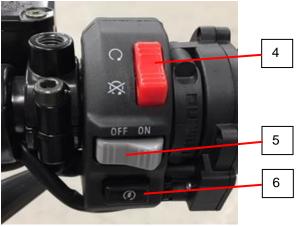

Controls - Right handlebar switches

- 4. Main power / emergency button
- 5. Driving sound forward on / off
- 6. Start/Stop button



Right handlebar

- 7. Direction indicator
- 8. Buttons
- 9. Total driven and trip km indicator
- 10. Verlichting indicator
- 11. Battery indicator
- 12. Speed

* Pay attention! Standard leave the parking brake in unlocked position. Only change the lever to locked position with an emergency situation. Never drive with a manually released brake, this can cause damage to brake and motor.

Boot up

- 1. Get on the Virto2
- 2. Switch main power (4) on.
- 3. Press the start button (6).
- Unlock the Virto 2 with a password or with the bluetooth key.

Parking

- 1. Park the Virto2 on a flat surface.
- 2. Turn Virto2 off (6).
- 3. Switch main power (4) off.

Drive

- 1. Keep both hands on the handlebars.
- 2. Accelerate slowly to the desired speed.
- 3. Lean along in the turns with an appropriate speed.
- 4. Always brake with both brakes.



The Virto2 may be loaded up to a maximum of 125 kg, this includes the driver. The maximum weight for in the luggage compartment and on the luggage carrier is 15 kg.



In an emergency, the main power switch (4) can be used as an emergency brake while driving. The total Virto2 is switched off and the engine brake is activated immediately. Pay attention! Use this **only** in an emergency.



Do not drive in puddles and avoid roads and slopes with an angle greater than + 10%. Avoid obstacles, they can damage the Virto2.

Battery load



Charging with included charger

- 1. Main power switch (4) OFF
- Plug in the power cord LED-1 lights up orange
- 3. Connect plug A to the Virto2

(Disconnect in reverse order)

LEDS on standard battery charger

LED 1- Orange - Connected to AC power

LED 1- Orange - Loading in progress

LED 2- Green - Fully charged



When the battery indicator shows only 2 lines, it must be charged a.s.a.p. Do **not** continue driving until the Virto2 automatically turns off, as this can damage the batteries.



Only charge the Virto2 with the supplied battery charger.